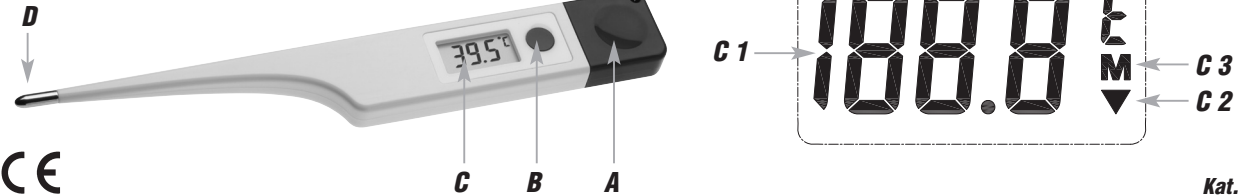
Bedienungsanleitung
Instruction manual
Mode d'emploi

VET 12
Digitales Veterinär-Thermometer

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03/25

VET 12
Digital veterinary thermometer

1. **Before you use this product**
 - Please make sure you read the instruction manual carefully.
 - Following and respecting the instructions in your manual will prevent damage to your instrument and loss of your statutory rights arising from defects due to incorrect use.
 - We shall not be liable for any damage occurring as a result of non-following of these instructions. Likewise, we take no responsibility for any incorrect readings or for any consequences resulting from them.
 - Please pay particular attention to the safety notices!
 - Please keep this instruction manual safe for future reference.
2. **For your safety**
 - This product is intended exclusively for rectal measurement of the body temperature of animals. It should only be used as described within these instructions.
 - Check the thermometer for any signs of damage or wear before use. Use the unit only when it is perfectly fine.
 - Unauthorized repairs, alterations or changes to the product are prohibited.
 - Operation in strong electromagnetic fields, such as next to a mobile phone, can lead to malfunction.
 - Always discuss the determined values with your veterinarian.

**Caution!
Risk of injury:**

- The instrument contains small parts that could be swallowed. Keep this device and the battery out of reach of children and animals.
- Batteries contain harmful acids and may be hazardous if swallowed. If a battery is swallowed, this can lead to serious internal burns and death within two hours. If you suspect a battery could have been swallowed or otherwise caught in the body, seek medical help immediately.
- Batteries must not be thrown into a fire, short-circuited, taken apart or recharged. **Risk of explosion!**
- Low batteries should be changed as soon as possible to prevent damage caused by leaking.
- Avoid contact with skin, eyes and mucous membranes when handling leaking batteries. In case of contact, immediately rinse the affected areas with water and consult a doctor.

Important information on product safety!

- Do not expose the device to extreme temperatures, vibrations or shocks. Do not bend the instrument.

3. Elements and buttons

- | | |
|-------------------------------|---------------------------------------|
| A: Battery compartment | B: ON/OFF button |
| C: Display | C 1: Temperature display |
| C 2: Battery symbol | C 3: Symbol measurement memory |
| D: Temperature probe | E: Hole for fixing cord |

4. Use

- Disinfect the tip before each use!
- Press the ON/OFF button.
- When the unit is switched on, a signal tone sounds and all segments are briefly displayed.
- The latest measured temperature (M) will appear shortly on the display. The instrument starts a self-test, then 37 °C and Lo °C is shown.
- If the room temperature is higher than 32 °C, the room temperature would be displayed instead of Lo °C. °C starts flashing. The device is now ready for use.
- Place the thermometer at the site of measurement.
- The actually measured maximum temperature is permanently shown on the display and °C will be flashing.
- If the maximum temperature remains stable, °C is no longer flashing and an alarm tone will sound.
- Remove the thermometer. Press the ON/OFF button to switch off the device. The device will turn off automatically after approx. 9 minutes.

5. Care and maintenance

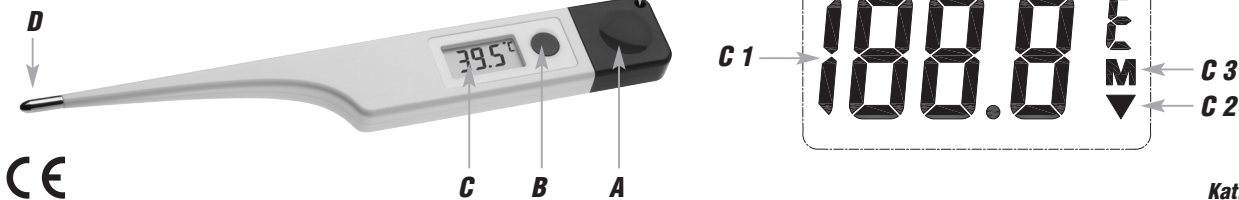
- Clean the instrument before and after each use with a soft damp cloth. For disinfection please use water-diluted isopropyl alcohol according to the manufacturer's instructions. Any other cleansing substances or cleansing method can cause dysfunction or damage to the instrument. Do not immerse in water or sterilize by boiling method, gas or steam autoclaves.
- Remove the battery if you do not use the device for a long period of time.
- Store the instrument in its case in a dry place.

5.1 Battery replacement

- Change the battery when the display starts to fade or the battery symbol "▼" appears on the display.
- Open the battery compartment at the back with a suitable screwdriver.
- Remove carefully the battery with a sharp object and insert a new battery LR 41 (+ pole above). Make sure the polarity is correct.
- Close the cover carefully, paying special attention to the sealing ring.

Bedienungsanleitung
Instruction manual
Mode d'emploi

TFA®

Kat. Nr. 15.2022

VET 12
Digital veterinary thermometer

GB

6. Troubleshooting

Problem	Solution
No display	→ Switch on the device → Ensure that the battery polarity is correct (+pole above) → Change the battery
Incorrect indication	→ Change the battery → Check the correct position of the thermometer → Remove the battery Wait one minute and then reinsert the battery

7. Waste disposal

Dispose of the packaging in an environmentally friendly manner using the collection systems that have been set up.

Disposal of the electrical device

Remove non-permanently installed batteries and rechargeable batteries from the device and dispose of them separately. This product is labelled in accordance with the EU Waste Electrical and Electronic Equipment Directive (WEEE). This product must not be disposed of in ordinary household waste. As a consumer, you are required to take end-of-life devices to a designated collection point for the disposal of electrical and electronic equipment, in order to ensure environmentally-compatible disposal. The return service is free of charge. Observe the current regulations in place!

Disposal of the batteries

Batteries and rechargeable batteries must never be disposed of with household waste. They contain pollutants such as heavy metals, which can be harmful to the environment and human health if disposed of improperly, and valuable raw materials such as iron, zinc, manganese or nickel that can be recovered from waste. As a consumer, you are legally obliged to hand in used batteries and rechargeable batteries for environmentally friendly disposal at retailers or appropriate collection points in accordance with national or local regulations. The return service is free of charge. You can obtain addresses of suitable collection points from your city council or local authority.

The names for the heavy metals contained are:
Cd=cadmium, Hg=mercury, Pb=lead.

WARNING!
Damage to the environment and health through incorrect disposal of the batteries!

8. Specifications

Type	Maximum thermometer
Measuring range Lo °C	32 °C ... 42.9 °C Surrounding temperature is lower 32 °C
Hi °C	Surrounding temperature is higher than 42.9 °C
Accuracy	±0.1 °C at 35.0 °C...42 °C ±0.2 °C otherwise
Resolution	0.1 °C
Measurement time (rectal)	Approx. 10-20 seconds
Storage conditions	Temperature -10 °C...+40 °C Humidity 30...90 % rH
Ambient conditions by use	Temperature 10 °C...35 °C Humidity 40...80 % rH
IP protection class	IP67
Battery	1x Button cell battery LR 41 1.5 V (included)
Brand name Manufacturer	New Leader NEWLEADER BATTERY INDUSTRY (DEQING) CO, LTD, No.108 Chaoyang West Road, Deqing, Zhaoqing, Guangdong, China
Housing dimension	188 x 26 x 11 mm
Weight	23 g

TFA Dostmann GmbH & Co.KG, Zum Ottersberg 12, 97877 Wertheim, Germany
No part of this manual may be reproduced without written consent of TFA Dostmann. The technical data are correct at the time of going to print and may change without prior notice.
The latest technical data and information about this product can be found in our homepage by simply entering the product number in the search box.

www.tfa-dostmann.de
E-Mail: info@tfa-dostmann.de

03/25

Instruction manuals

www.tfa-dostmann.de/en/service/downloads/
instruction-manuals

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